Program name: CERAMOTION ZHT MS

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Download programs and import them into the controller:

- Always perform a service export to a USB flash drive before importing programs.
 - Insert an USB flash drive into the controller. Then, depending on the controller, press "Service Export" or select "Service Export" from the Info menu to back up your old programs. Wait until the writing process is complete.
 - If the USB flash drive is not recognized, try a different type.
- Download the desired programs and copy them to the USB flash drive to a folder called "Import".
- For controllers Series 500 from V 2.26 (Info menu) onwards, the program can be imported easily. The program is then automatically copied to a free program slot.
- For Series 400 controllers (all versions) and Series 500 controllers up to V 2.25 (Info menu), the program name of the downloaded program contains a target program number. The program "Prg.01.xml" would therefore overwrite program number 01 when imported. If this is not

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desired, please change the file name so that the program is transferred to a free program number. For example, if you want to copy the program to program number 02, change the file name from "Prg.01.xml" to "Prg.02.xml." You can then import the program file. If there is no free program slot available, please delete a program that you no longer need.

- Import the program into the controller
 - Log in as administrator (see controller manual)
 - Choose "Settings" -> Import/Export -> Import Programs -> Yes

Segment	Start	Target	Time	Rate	Predrying	Cooling
1	30°C	1200°C		8°C/min	0	0
2	1200°C	1200°C	30 min		0	0
3	1200°C	1300°C		2°C/min	0	0
4	1300°C	1530°C		4°C/min	0	0
5	1530°C	1530°C	120 min		0	0
6	1530°C	800°C		8°C/min	0	0
7	800°C	0°C			0	0